

Luna GII Quick Operation Manual

1. Button Functions



Mute Button: Press it to mute alarms




Knob: Start treatment and adjust device settings



Ramp Button: Press it to enable the Ramp feature

2. Setting up the Device

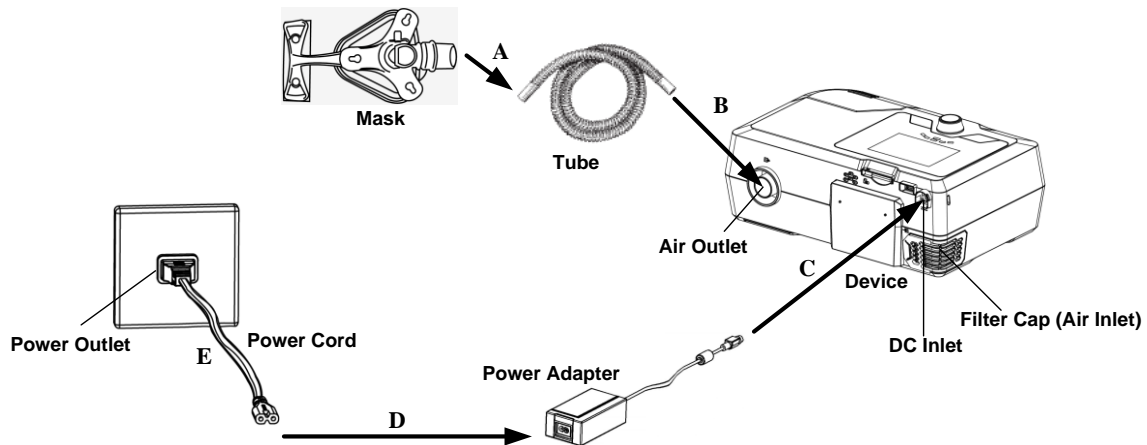

A. Connect one end of the tube to the assembled mask.

B. Connect the other end of the tube securely to the air outlet of the device, as shown in the figure above.

C. Connect the power adapter to the DC inlet of the device.


D. Connect the power cord to the power adapter.


E. Plug the power cord into the power outlet.

3. Adjusting Device Settings


A. After the device is connected to power, its screen displays the Main Interface shown in the figure below.




B. From the Main Interface shown above, press and hold the **Ramp Button**  for three seconds. The screen displays the Initial Setup Interface, as shown in the figure below.

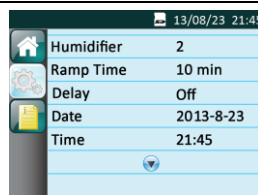


C. Press the **Knob**  to access the Setup Interface, as shown in the figure below.

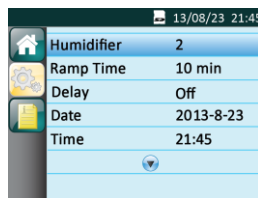

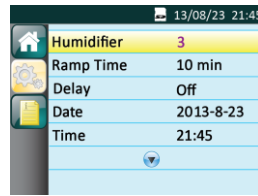


D. Press and turn **the Knob**  to adjust an option.

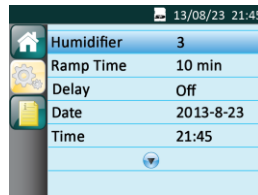


E. Press **the Knob**  to confirm your adjustment to the option.



F. The cursor switches among the options as you turn **the Knob** . Refer to steps C-E to adjust other options. When the cursor is on the **Home** option, as shown in the figure below, press **the Knob**  to exit the Setup Interface.

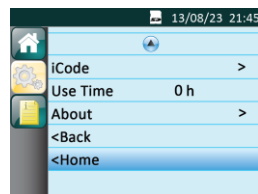

4. Starting Treatment

After you finish adjusting device settings and return to the Main Interface, press **the Knob**  to start treatment.